

# Hydrocollator Heating Unit Cleaning Schedule

---

## DAILY

- Monitor that the water temperature maintains a therapeutic range of  $160^{\circ} \pm 5^{\circ}$ .
- Record the temperature on the daily log.

## WEEKLY

- Clean the outside of the hydrocollator.

## MONTHLY

### Steps for Cleaning the Hydrocollator Unit:

- Un-plug the unit and letting it cool.
- Place the cooled "hot packs" on a towel.
- Check the hot packs for any damage or wear.
- Report any damage or wear to the Administrator for replacement.
- Drain the unit.
- Clean the unit by wiping the unit and remove any stains with stainless steel polish.
- Refill the unit with water.
- Place the hot packs back into the unit.
- Plug the unit in and reheat the water and hot packs.